

HARTMANN




Four-point plan for correct fitting of MoliCare® all-in-one briefs


1 Measure the waist and hips and, using the largest of these two measurements, ensure the correct product is selected. If you are 'between sizes', always start with the smaller size. Open up the **MoliCare** brief lengthways, while leaving the non-woven side panels in the folded position. Hold the product at each end and gently stretch the product lengthways once. This helps to activate the inner cuffs and creates an anatomical 'cupped' shaped for fitting.



2 Place the 'cupped' **MoliCare** brief between the thighs from the front, with the soft white absorbent layer next to the skin and the absorbent area with tapes to the rear. Unfold the rear absorbent area and fit snugly to the body. The wetness indicator should be in line with the spine. At the front, centre the absorbent cushion in the crotch area.

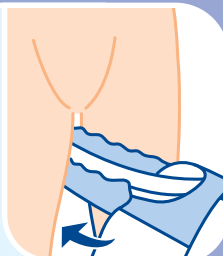

3 Unfold the front absorbent area and fit into position, making sure the product is fitted as **high as possible** in the groin area. Unfold the front side panels and fit snugly to the body. Unfold the side panels at the back of the product. To fasten the **MoliCare** brief, stretch and position the upper adhesive tapes at a slightly downwards angle onto either the non-woven side panel or waterproof backing of **MoliCare**. The lower adhesive tapes should stretch and point diagonally upwards.

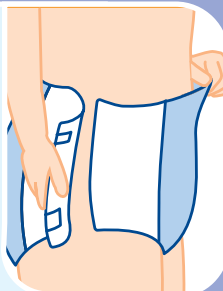

4 Finally, check that the **MoliCare** brief is comfortably and closely fitted to the body. The striped design helps you fit **MoliCare** as symmetrically as possible. When the stripes are aligned, the super-absorbent core is correctly positioned around the crotch area. To open and refasten the tapes, open the white tape only and, when ready, refasten the white tape back onto the blue tape, which remains attached to the front of **MoliCare**.





MoliCare®

MoliCare all-in-one briefs are designed for clients with severe urinary and/or faecal incontinence. The breathable non-woven side panels and elasticised fixation tapes of MoliCare ensure optimal client comfort.*

* These unique features are available in the small, medium and large sizes of MoliCare and MoliCare Super only

MoliCare

Size	Waist/Hip measurement	Case quantity	Product code	ISO Absorbency
extra small	40/70 cm	3 x 28	169 213	1621 ml
small	60/90 cm	3 x 28	169 961	1484 ml
medium	90/120 cm	3 x 28	169 962	2070 ml
large	120/150 cm	3 x 28	169 963	2395 ml
extra large	140/175 cm	4 x 14	169 901	4119 ml

MoliCare super

small	60/90 cm	3 x 28	169 971	1747 ml
medium	90/120 cm	3 x 28	169 972	2800 ml
large	120/150 cm	3 x 28	169 973	3020 ml

Molicare Super Plus

small	50/80 cm	4 x 14	169 472	2143 ml
medium	70/120 cm	4 x 14	169 672	3508 ml
large	100/150 cm	4 x 14	169 872	4598 ml

International Standard Method ISO 11948-1

The use of oil-based or silicone-based creams, or talcum powder on the skin, may adversely affect the permeability of absorbent continence care products.



PAUL HARTMANN Limited, Heywood, Lancashire OL10 2TT
 Tel: 01706 363200 Fax: 01706 363201 www.hartmann.co.uk

LIT(UK)0173